

Archtek Press and Fit Professional Tooth Whitening System

Your solution to whiter, brighter teeth in days not weeks! Our safe and effective ingredients work in just 30 minutes a day for five days, with no overnight whitening required. The Press and Fit trays provide maximum whitening, while lessening the possibility of gum irritation.

THREE STEPS TO A PERFECT WHITE SMILE

Repeat this process once/day for 5 days.

READY: Form your Press and Fit wax bleaching trays according to instructions. Brush teeth lightly and rinse your mouth with water before each whitening application.

AIM: Run a small amount of flavored gel along the lower frontal portion of each Press and Fit wax tray. Avoid overfilling trays with gel; a small amount will deliver maximum whitening power.

WHITEN: Carefully place gel filled upper and lower trays over teeth, close mouth. Avoid swallowing excess gel. Remove trays after 30 minutes. Brush or rinse remaining gel away from teeth and gums. **CAUTION: Do not wear while sleeping!** Gently rinse trays with cool tap water being careful not to deform the tray and store in protected case. Each tray can be used for approximately 2 to 3 whitening applications.

Dentists recommend the use of peroxide containing products for whitening teeth. Clinical studies suggest that you can expect significant shade changes in just 5 days. Your results may vary due to the nature of the discoloration found in your teeth. If teeth are discolored as a result of tetracycline, fluorosis or more difficult stains, trays may be worn for a longer period of time upon your dentist's recommendation.

As always, it is recommended that users of whitening products see their dental professional on a regular basis for check-ups and discuss the use of any whitening products with their dentist.

PRECAUTIONS

- Keep out of reach of children. Not recommended for use by children under the age of 14.
- Trays should be worn for 30 minutes. Some tingling in the gums may result from excessive use without additional whitening benefit.
- Not recommended for use by women who are pregnant or nursing.
- Bleaching materials contain peroxides and will not change the appearance of fillings, crowns, or veneers.
- Discontinue use if pain or severe sensitivity develops in the teeth or gums and contact your dentist.
- If gel comes in contact with eyes, wash with copious amounts of water.
- Patients allergic to peroxides should not use this product.
- Avoid getting whitening gel on clothing. Peroxides may bleach materials.
- This product should not be used if you have dental decay, abscessed teeth or untreated gum disease or are wearing braces.
- For optimal results, avoid using staining agents such as coffee, tea, red wine, grape juice and tobacco during the treatment period.



FITTING INSTRUCTIONS

- Remove a wax tray from the case and place in the mouth on the tongue with lips closed for 10 seconds being careful not to deform the tray. This allows the wax to warm to the body temperature and become more flexible
- Center the scalloped top edge of the wax tray along the line at the top of the teeth where tooth enamel meets the gum tissue
- Gently fold the wax back along the front of the teeth covering all enamel surfaces but not covering the gum tissue
- Gently and slowly fold the bottom of the wax tray around to the back side of the teeth making sure to keep the front position from moving out of place.
- Once firmly folded to the contour of the teeth, carefully press fingers against the wax to imprint the anatomy of the teeth into the wax.



- Gently remove the tray, place bleaching solutions into the wax tray and place the loaded tray over the teeth.

CAUTION

- Do not bite into the wax tray
- The wax tray can be reused with careful use. Rinse in clean water after each use.
- Keep the trays away from excessive heat and out of direct sunlight. Do not sleep with the tray in the mouth.

WARNINGS

Store in a cool dry place. Keep away from heat and direct sunlight. Keep out of reach of children. Not recommended for use by children under the age of 14.

Archtek
INNOVATION IN DENTISTRY

12105 West Cedar Drive
Lakewood, Colorado 80228
800/763-8129 ■ www.archtekinc.com

PRESS AND FIT PROFESSIONAL

Tooth Whitening System

KIT INCLUDES

- 12 Press and Fit Wax Bleaching Trays
- Two 3ml syringes of Peroxide Whitening Gel
- Vented 7-inch storage case with a mirror

INGREDIENTS: Whitening Gel contains wax, carbamide peroxide, water, glycerin, sodium saccharin, artificial flavor.

Gel made in USA • Wax Trays made in Taiwan

Archtek
INNOVATION IN DENTISTRY